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The screenshot shows the 'CMDfact Patientenverwaltung' window. It is divided into two main sections: 'Patientinnen und Patienten' on the left and 'Untersuchungen' on the right. In the 'Patientinnen und Patienten' section, there is a search bar with 'ma' entered and a 'Neu einlesen' button. Below it is a table with two columns: 'Name' and 'Pat.-Nr.'. The table contains two entries: 'Mustermann, Martina' with ID '123456' and 'Musterfrau, Mara' with ID '133578'. A red circle with the number '1' points to the second entry. Below the table is a button labeled 'Diesen Patienten auswählen'. In the 'Untersuchungen' section, there is a search bar with '123456 |' and a table with three columns: 'Datum', 'Untersuchung', and 'Nr.'. The table contains five entries, all dated '21.06.2017', with various examination types and numbers. Below the table are two buttons: 'Neuer Patient' and 'Patient Bearbeiten'. A red circle with the number '3' points to the 'Patient Bearbeiten' button. At the bottom of the window, there is a status bar with 'Patientin/Patient Pat.-Nr. - - -' on the left and 'Praxis CMD-Centrum Hamburg-Eppendorf' on the right.

Name	Pat.-Nr.
Mustermann, Martina	123456
Musterfrau, Mara	133578

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The screenshot shows the 'CMDfact Patientenverwaltung' window. It is divided into two main sections: 'Patientinnen und Patienten' on the left and 'Untersuchungen' on the right. The left section features a search bar (labeled 'Suche') with the placeholder text 'Pat.-Nr, Name oder Teil davon' (labeled 2), a 'Neu einlesen' button, and a list of patients. The first patient, 'Mustermann, Martina' with ID '123456', is selected (labeled 1). Below the list is a button 'Diesen Patienten auswählen'. The right section, titled '123456 | Mustermann, Martina', displays a table of examinations (labeled 3) with columns for 'Datum', 'Untersuchung', and 'Nr'. The table contains five entries for the date 21.06.2017. At the bottom of the window, there are buttons for 'Neuer Patient' and 'Patient Bearbeiten'. The status bar at the bottom shows 'Patientin/Patient Pat.-Nr. - - -' and 'Praxis CMD-Centrum Hamburg-Eppendorf'.

Name	Pat.-Nr.
Mustermann, Martina	123456
Musterfrau, Mara	133578
Musterpatient, Paul	999781
Patient01, Patient1	1
Patient02, Patient2	2
Patient03, Patient3	3
Patient04, Patient4	4
Patient05, Patient5	5
Patient06, Patient6	6
Patient07, Patient7	7

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The screenshot displays the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below this is the 'CMDfact' logo and the 'dentaConcept' logo. A horizontal bar contains several module icons: 'CMD CHECK', 'CMD brux', 'CMD status', 'CMD manu', 'CMD Pain', 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arzt-brief'. The 'CMD Pain' icon is highlighted with a red circle and a blue circle, and a hand icon is shown clicking it. Below this bar is a large white area with a vertical sidebar on the left containing four colored icons (purple, red, orange, yellow). The purple icon is highlighted with a red circle and a blue circle, and a hand icon is shown double-clicking it. A mouse cursor is also shown over the purple icon. At the bottom of the sidebar, there is a 'Tag 1' label with a calendar icon. A large grey arrow points to the right across the bottom of the main area. At the very bottom, there is a footer with 'Patientin/Patient Pat-Nr. ---' on the left and 'Praxis CMD-Centrum Hamburg-Eppendorf' on the right.

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The screenshot shows the CMDfact 4 software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A row of icons represents various functions: a person icon, 'CMD CHECK', 'CMD brux', 'CMD status', 'CMD manu', 'CMD pain', 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arztbrief'. The main area features a vertical timeline with five colored blocks (grey, purple, red, orange, yellow). The bottom block is labeled 'Tag 1' and has a calendar icon. A calendar pop-up is open, showing 'July 2020' with the 12th highlighted. A red circle '2' points to the calendar, and another red circle '1' points to the calendar icon on the timeline. At the bottom, there is a status bar with 'Patientin/Patient Pat-Nr. ---' on the left and 'Praxis CMD-Centrum Hamburg-Eppendorf' on the right.

Mo	Tu	We	Th	Fr	Sa	Su
		1	2	3	4	5
6	7	8	9	10	11	12
13	14	15	16	17	18	19
20	21	22	23	24	25	26
27	28	29	30	31		

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The screenshot shows the CMDfact 4 software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A row of colorful icons represents different modules: CMD CHECK (yellow), CMD brux (orange), CMD status (red), CMD manu. (dark red), CMD pain (purple), CMD tomo (blue), CMD trace (light blue), CMD 3D (dark blue), Diagnose Pilot (green), and CMD Arzt-brief (grey). Below the icons is a vertical timeline with five colored blocks (grey, purple, red, orange, yellow) representing different modules. A grey arrow at the bottom of the timeline points to the right and is labeled 'Tag 1' with a calendar icon. A context menu is open over the purple block, showing two options: 'Formblatt ausdrucken' and 'Untersuchung löschen'. A red circle with the number '1' is placed above the menu, and a red circle with the number '2' is placed below it. The bottom status bar contains 'Patientin/Patient Pat-Nr. ---' on the left and 'Praxis CMD-Centrum Hamburg-Eppendorf' on the right.

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The screenshot displays the CMDfact 4 software interface. At the top, there is a menu bar with 'Datei' and 'Extras'. Below it, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A row of icons represents various modules: CMD CHECK, CMD brux, CMD status, CMD manu, CMD pain, CMD tomo, CMD trace, CMD 3D, Diagnose Pilot, and CMD Arzt-brief. The main content area shows a 'CMDstatus' window titled 'Palpationsbefunde' for patient 'Mara Musterfrau' (Pat-Nr. 133578). The window contains two anatomical diagrams of a human head and neck, one labeled 'rechts' (right) and one 'links' (left), with red highlights on the jaw and neck muscles. A table lists various muscles and symptoms with checkboxes for 'Schmerz' (pain) and 'Notizen' (notes). The table is as follows:

	Schmerz	Notizen
Missempfindung	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. masseter superfic.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. masseter prof.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Kiefergelenk lateral	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Kiefergelenk dorsal	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. temporalis ant.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. temporalis med./post.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Suboccip.-/Nacken-M.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. trapezius	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. sternodeidomastoideus	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
infrahyoidale M.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
suprahyoidale M.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. pterygoideus med.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. digastricus venter post.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
M. pterygoideus lat.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>

At the bottom of the window, it says 'Untersuchung vom 16.07.2017' and 'Praxis'. A red arrow points to the 'X' button in the window title bar, which is circled in red. Another red circle highlights the 'X' button in the background window's title bar.

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The screenshot shows the CMDstatus software interface. At the top, there are two window titles: "CMDfact" and "CMDstatus". Below the window titles is a menu bar with "Datei" and "?". The main header features the "CMDstatus" logo and the "dentaConcept" logo. A row of icons represents different examination modules: Stress, Ortho, Para, Palpation, Geräusche (highlighted), Mobilität, Okklusion1, Okklusion2, and Reaktion. Below the icons is a list of examination categories, with "Animation: Diskusverlagerung mit Reposition" selected. A text box provides a detailed explanation of this condition. To the right, a video player shows an anatomical illustration of a joint with a blue disc. Below the video player, there is a section titled "Kiefergelenk-Geräusche" and "Diskusverlagerung mit Reposition". At the bottom of the interface, there is a patient information bar with "Patientin/Patient: Maria Mustermann" and "Pat-Nr.: 133378", and a practice information bar with "Praxis: CMD-Centrum Hamburg-Eppendorf".

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The screenshot shows the CMDstatus software interface. At the top, there are two window titles: "CMDfact" and "CMDstatus". Below the titles is a menu bar with "Datei" and "?". The main header features the "CMDstatus" logo and the "dentaConcept" logo. A toolbar contains several icons representing different functions: Stress, Ortho, Para, Palpation, Geräusche (highlighted), Mobilität, Okklusion1, Okklusion2, and Reaktion. The main content area is divided into a left sidebar and a right main panel. The sidebar contains a list of animation options under the heading "GELENKGERÄUSCHE":

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The right main panel displays a 3D anatomical model of a joint with a blue disc. Below the model is a video player with a play button (1) and a full-screen button (2). A keyboard inset shows the Esc key (3) and the "Anleitung" button on the right side of the interface (3). The patient information at the bottom reads: "Patientin/Patient: Maria Mustermann Pat-Nr.: 133578".

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The screenshot displays the CMDstatus software interface. At the top, there are two window titles: 'CMDfact' and 'CMDstatus'. The main header features the 'CMDstatus' logo and the 'dentaConcept' logo. Below the header is a row of nine red icons representing different modules: Stress, Ortho, Para, Palpation, Geräusche, Mobilität, Okklusion1, Okklusion2, and Reaktion. The 'Palpation' module is currently selected. A toolbar below the icons contains various editing tools like cut, copy, paste, undo, redo, and text formatting options (bold, italic, underline, text color, background color). The main text area shows a dropdown menu for 'Absatz' (paragraph), a font family dropdown set to 'Segoe UI', and a font size dropdown set to '24 Pt'. The text in the main area reads: '-- Notizen zu Palpationsbefunde' (highlighted with a red circle '2'), followed by 'Hier könnte eine Notiz stehen.' and a large, bold, black text: 'Sie können Sie formatieren | wie sie wollen.' (highlighted with a red circle '1'). On the right side of the interface, there is a vertical sidebar with a 'Notizen' button (highlighted with a red circle '1') and an 'Anleitung' button. At the bottom of the interface, there is a patient information bar showing 'Mara Musterfrau', 'Pat-Nr. 133578', '232 Zeichen', and a checkbox for 'Notizen auf neue Seite drucken'. The footer of the application shows 'Patientin/Patient Mara Musterfrau Pat-Nr. 133578' and 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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The screenshot displays the 'CMDfact' software interface. At the top, there is a menu bar with 'Datei' and 'Extras ?'. The main header features the 'CMDfact' logo and the 'dentaConcept' logo. Below the header, there are several icons for different modules: 'CMD CHECK', 'CMD BRUX', 'CMD status', and 'CMD WA'. The central part of the screen shows a printed report for 'Mara Mustermann', dated '27.02.2006'. The report is titled 'Klinischer Funktionsstatus (4)' and includes various sections: 'Stress', 'Orthopädisches Screening', 'Palpationsbefunde', 'Mobilität', 'Okklusionsbefunde 1', 'Okklusionsbefunde 2', 'Reaktionstests', and 'Notizen'. Each section contains checkboxes and text boxes for recording findings. The 'Palpationsbefunde' section includes diagrams of the head and neck with red highlights indicating areas of concern. The 'Okklusionsbefunde 1' section includes a grid for 'Offnung UK' and 'Overbite (vert.)'. The 'Reaktionstests' section includes a 'Provokationstest (Klopp-Poulsen)' and a 'Resilienztest nach GERBER'. The 'Notizen' section contains a note: 'Notizen werden auf zusätzlichem Blatt gedruckt...'. The bottom of the screen shows the patient information: 'Patientin/Patient Pat.-Nr. ---' and the version number 'Version 4.0, Hamburg 1996-2016'.

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The screenshot shows the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left, and the 'dentaConcept' logo is on the right. A red circle with the number '1' highlights the 'CMDfact' logo. Below the logos, there is a row of icons for various functions: 'CMD CHECK' (highlighted with a dashed yellow box and a yellow arrow pointing to the main screen), 'CMD brux', 'CMD status', 'CMD manu', 'CMD pain', 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arztbrief'. The main content area displays a large orange box with the text 'CMDCHECK[✓] Version 4 4.20' and an illustration of a human head in profile with red dots on the ear and neck. At the bottom of the main area, there is a date '16.07.2017' and a 'CMD CHECK' button. Below the main area, there is a footer with patient information: 'Patientin/Patient **Mara Musterfrau** Pat-Nr. 123456' and 'Praxis **CMD-Centrum Hamburg-Eppendorf**'.

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The screenshot shows the CMDfact 4 software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A toolbar contains several icons: a person icon, 'CMD CHECK', 'CMD BRUX', 'CMD status', 'CMD manu', 'CMD pain', 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arzt-brief'. The main area is mostly obscured by a dialog box titled 'Keine Vollversion'. The dialog box contains the following text: 'Fehler beim Öffnen der Datenbank' (Error opening the database), 'Die externe Datenbank kann nur mit der Vollversion von CMDcheck 4 verwendet werden.' (The external database can only be used with the full version of CMDcheck 4), and an 'OK' button. A red circle with the number '1' is drawn around the 'OK' button. In the bottom left corner of the main window, there is a 'CMD CHECK' icon and a date field showing '16.07.2017'. At the bottom of the window, the patient information is displayed: 'Patientin/Patient **Mara Musterfrau** Pat-Nr. 123456' and 'Praxis **CMD-Centrum Hamburg-Eppendorf**'. A large grey arrow points from the 'CMD CHECK' icon in the bottom left towards the right side of the screen.

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The screenshot shows the 'CMDcheck 4' application window. The title bar reads 'CMDfact' and 'CMDcheck 4'. The main header features the 'CMD CHECK' logo and the 'dentaConcept' logo. Below the header is a navigation bar with icons for 'Patienten', 'CMD-Kurzbefund' (selected), 'Befund exportieren', and 'Befund drucken'. The main content area is divided into two columns. The left column, titled 'CMD-Kurzbefund', contains a list of symptoms with checkboxes: 'Mundöffnung asymmetrisch' (checked), 'Mundöffnung eingeschränkt' (unchecked), 'Gelenkgeräusche' (checked), 'Okklusale Geräusche' (checked), 'Palpation schmerzhaft' (checked), and 'Exzentrik traumatisch' (unchecked). The right column, titled 'CMD ist...', shows a legend with three colored circles: a red circle for 'sehr wahrscheinlich', a yellow circle for 'wahrscheinlich', and a green circle for 'unwahrscheinlich'. Below the legend are two buttons: 'Sichern' and 'Export + Programm schließen'. The 'Export + Programm schließen' button is highlighted with a blue border and a red circle containing the number '1'. At the bottom of the window, patient information is displayed: 'Patient/in Mara Musterfrau Pat-Nr. 123456' and 'Untersuchung vom 11.05.2018 (gesichert)'. The bottom right corner shows the practice name: 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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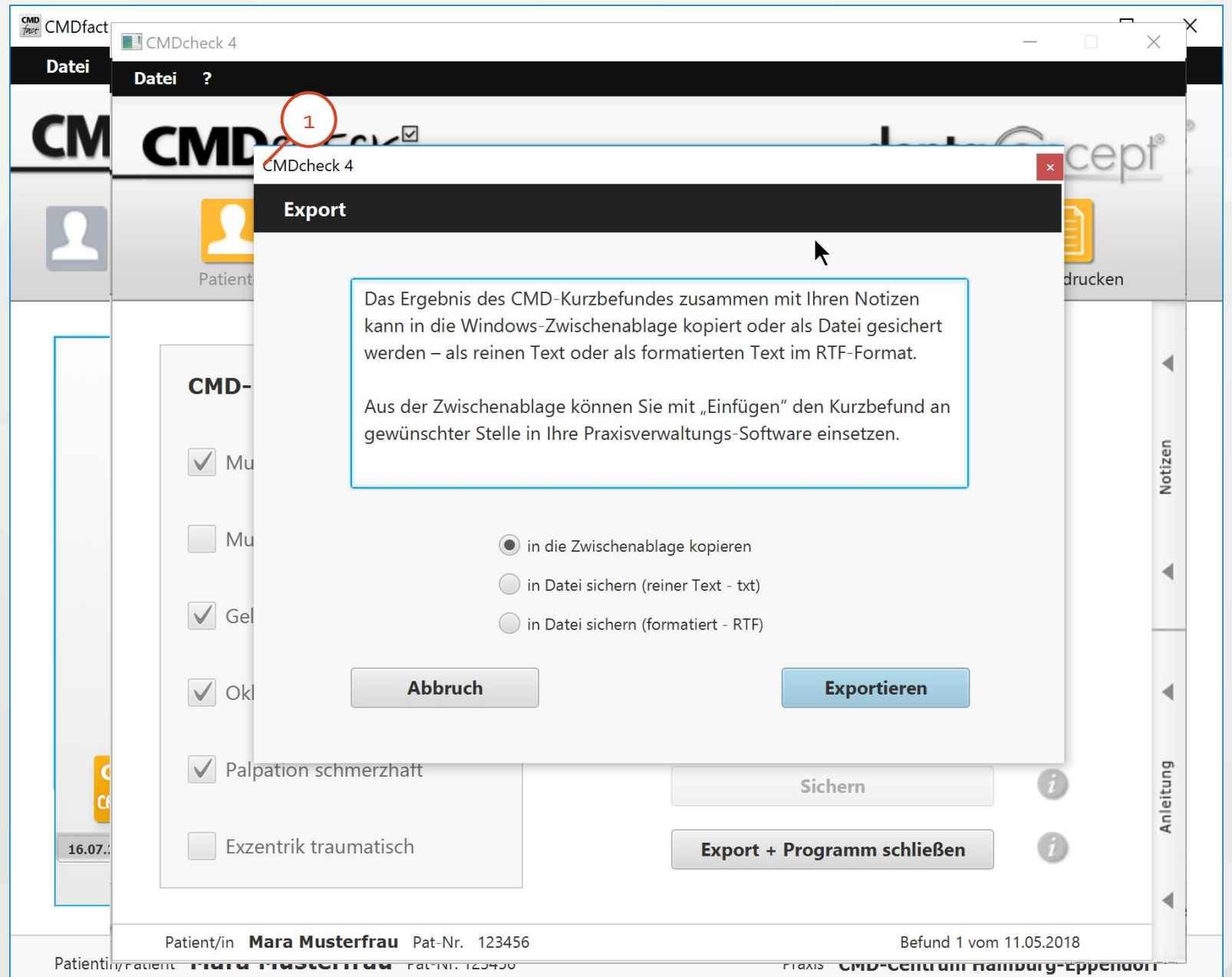
The screenshot displays the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A navigation bar contains several icons: a person icon, 'CMD CHECK', 'CMD brux', 'CMD status' (highlighted with a red dashed box and a red circle containing the number '1'), 'CMD manu', 'CMD pain', 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arzt-brief'. The main content area features a large red banner for 'CMDstatus Version 4 4.10' with an illustration of a human head and neck. A red arrow points from the 'CMD status' icon in the navigation bar to a smaller 'CMD status' icon in the bottom left corner of the main area. Below the banner, there are two smaller icons: 'CMD status' and 'CMD CHECK'. The date '16.07.2017' is displayed in the bottom left corner of the main area. At the bottom of the window, the patient information 'Patientin/Patient **Mara Musterfrau** Pat-Nr. 133578' is shown on the left, and the practice information 'Praxis **CMD-Centrum Hamburg-Eppendorf**' is shown on the right.

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The screenshot shows the CMDstatus software interface. At the top, there is a menu bar with 'Datei' and a question mark. Below the menu bar, the 'CMDstatus' logo is on the left and the 'dentaConcept' logo is on the right. A navigation bar contains several icons representing different assessment areas: Stress (lightning bolt), Ortho (head with arrow), Para (knee), Palpation (head profile), Geräusche (ear), Mobilität (spine), Okklusion1 (teeth), Okklusion2 (teeth), and Reaktion (crossed arms). The 'Stress' icon is highlighted. A red circle with the number '1' points to the 'CMDstatus' logo. Below the navigation bar, the 'Stress' section is displayed. A red circle with the number '2' points to the 'Übertrag' text. The 'Stress' section has a title 'Stress' with a square icon. Below the title, it says 'Übertrag aus dem Fragebogen Stressbelastung:' followed by a text input field containing 'hier eintragen' and the label 'Punkte'. Below this, it says 'Die Stressbelastung ist' followed by three radio button options: 'extrem (> 200 Punkte)', 'erhöht (100-200 Punkte)', and 'normal (< 100 Punkte)'. At the bottom of the interface, patient information is displayed: 'Mara Musterfrau', 'Pat-Nr. 133578', and 'Untersuchung vom 16.07.2017'. The bottom right corner shows 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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The screenshot displays the CMDstatus software interface. At the top, there is a navigation bar with the logo 'CMDstatus' and 'dentaConcept'. Below the logo is a row of icons representing different clinical areas: Stress, Ortho, Para (highlighted with a red circle 1), Palpation, Geräusche, Mobilität, Okklusion1, Okklusion2, and Reaktion. The main content area is titled 'Parafunktion' (highlighted with a red circle 2) and contains a list of checkboxes for various symptoms. To the right of each checkbox is a text input field for specifying the affected teeth. The third item, 'Abrasionen/Attritionen', has a red circle 3 pointing to its text field. At the bottom of the interface, patient information is displayed: 'Mara Musterfrau', 'Pat-Nr. 133578', and 'Untersuchung vom 16.07.2017'. The footer also includes 'Patientin/Patient Mara Musterfrau Pat-Nr. 133578' and 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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The screenshot shows the 'CMDstatus' software interface. At the top, there are navigation icons for Stress, Ortho, Para, Palpation (highlighted), Geräusche, Mobilität, Okklusion1, Okklusion2, and Reaktion. The main window is titled 'Palpationsbefunde' and contains a table of findings for the right ('rechts') and left ('links') sides. The table has columns for 'Missempfindung' and 'Schmerz', each with checkboxes for '1' and '2'. The table lists various muscles and joints, with some cells containing '1' or '2' and others containing 'X'. To the left of the table is an anatomical diagram of the right side of the head and neck, and to the right is a diagram of the left side. Red shading on the diagrams indicates affected muscles. Three red circles with numbers 1, 2, and 3 are overlaid on the interface: circle 1 points to the 'Missempfindung' column header, circle 2 points to the 'Schmerz' column header, and circle 3 points to the red shading on the left-side anatomical diagram. At the bottom of the window, patient information is displayed: 'Mara Musterfrau', 'Pat.-Nr. 133578', and 'Untersuchung vom 16.07.2017'. The footer also includes 'Patientin/Patient: Mara Musterfrau Pat.-Nr. 133578' and 'Praxis: CMD-Centrum Hamburg-Eppendorf'.

	<input type="checkbox"/>	<input checked="" type="checkbox"/>	Missempfindung	Schmerz	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	
<input type="checkbox"/>	<input checked="" type="checkbox"/>		M. masseter superfic.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
<input type="checkbox"/>	<input checked="" type="checkbox"/>		M. masseter prof.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		Kiefergelenk lateral	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		Kiefergelenk dorsal	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		M. temporalis ant.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		M. temporalis med./post.	<input checked="" type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input checked="" type="checkbox"/>		Suboccip.-/Nacken-M.	<input checked="" type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input checked="" type="checkbox"/>		M. trapezius	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		M. sternodeidomastoideus	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		infrahyoidale M.	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		suprahyoidale M.	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		M. pterygoideus med.	<input type="checkbox"/>	<input type="checkbox"/>		
<input type="checkbox"/>	<input type="checkbox"/>		M. digastricus venter post.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		M. pterygoideus lat.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		

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The screenshot shows the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A row of icons represents different modules: 'CMD CHECK', 'CMD brux', 'CMD status', 'CMD manu', 'CMD pain', 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arztbrief'. The 'CMD manu' icon is highlighted with a red dashed box and a red circle containing the number '1'. A red arrow points from this icon to a large central window. This window has a dark red background and features a 3D anatomical illustration of a human head and neck with red spheres indicating points of interest. The text 'CMDmanu' and 'Version 4' is displayed above the large number '4.10'. The 'dentaConcept' logo is at the bottom right of this window. On the left side of the window, there are smaller icons for 'CMD manu', 'CMD status', and 'CMD CHECK'. A red circle containing the number '2' is positioned next to the 'CMD manu' icon. At the bottom of the software window, a status bar shows the date '16.07.2017'. Below the software window, the patient information 'Patientin/Patient **Mara Musterfrau** Pat-Nr. 133578' is displayed on the left, and the practice information 'Praxis **CMD-Centrum Hamburg-Eppendorf**' is displayed on the right.

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The screenshot shows the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A navigation bar contains several icons: a person icon, 'CMD CHECK', 'CMD BRUX', 'CMD status', 'CMD manu', 'CMD pain' (highlighted with a dashed box and a red circle with the number 1), 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot', and 'CMD Arztbrief'. The main content area features a large graphic of a human head in profile with red dots indicating pain points in the ear, jaw, and neck. To the right of the graphic, the text 'CMDpain Version 4 4.10' is displayed. Below the graphic, there is a vertical stack of buttons: 'CMD pain' (highlighted with a red circle with the number 2), 'CMD manu', 'CMD status', and 'CMD CHECK'. At the bottom left, the date '16.07.2017' is shown. At the bottom right, the 'dentaConcept' logo is visible. The footer contains the patient information: 'Patientin/Patient **Mara Musterfrau** Pat-Nr. 133578' and the practice name: 'Praxis **CMD-Centrum Hamburg-Eppendorf**'.

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The screenshot shows the CMDpain software interface. At the top, there are three main navigation buttons: 'Hauptbeschwerden', 'Schmerzregionen', and 'Verlauf'. Below these, the main content area is titled 'Schmerzzintensität der Hauptbeschwerden'. It features a horizontal slider scale from 0 to 10. The scale is labeled '0 = gar nicht' on the left and 'unerträglich = 10' on the right. A hand icon is shown interacting with the slider, with a red circle '2' indicating the movement. Below the slider is a text input field with the placeholder text 'Ich komme weil...'. At the bottom of the interface, there is a patient information bar showing 'Mara Musterfrau Pat.-Nr. 123456' and a date '16.07.2017'. A red circle '3' is placed over the date. The bottom right corner of the interface shows 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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The screenshot shows the CMDpain 4 software interface. The main window is titled 'CMDpain' and features the 'dentaConcept' logo. The interface is divided into three main sections: 'Hauptbeschwerden' (Main Complaints), 'Schmerzregionen' (Pain Regions), and 'Verlauf' (Course). The 'Hauptbeschwerden' section is active and displays a slider for 'Schmerzintensität der Hauptbeschwerden' (Pain intensity of main complaints). The slider ranges from 0 (gar nicht) to 10 (unerträglich), with a current value of 8. A hand icon is shown interacting with the slider. Below the slider is a text input field for 'Was sind Ihre Hauptbeschwerden?' (What are your main complaints?), containing the text 'Schmerzen im KG links und im Bereich unter dem Kinn'. The interface also includes a date field (16.07.2017), a patient name (Mara Musterfrau, Pat-Nr. 123456), and a checkbox for 'Werte anzeigen' (Show values), which is checked. The bottom of the screen displays the patient's name and ID (Mara Musterfrau, Pat-Nr. 133578) and the practice name (Praxis CMD-Centrum Hamburg-Eppendorf).

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The screenshot shows the CMDpain 4 software interface. The main window displays a pain assessment for a patient named Mara Musterfrau (Pat-Nr. 123456). The interface includes a navigation bar with three main sections: Hauptbeschwerden, Schmerzregionen, and Verlauf. The Schmerzregionen section is active, showing five pain regions with horizontal bar charts indicating the pain level (0 = gar nicht, 10 = unerträglich):

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The patient's name and ID are displayed at the bottom: Mara Musterfrau Pat-Nr. 123456. The investigation date is 16.07.2017. The software is from the Praxis CMD-Centrum Hamburg-Eppendorf.

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The screenshot shows the CMDpain 4 software interface. The main window displays a pain assessment form for patient Mara Musterfrau. The form includes a header with the logo 'dentaConcept' and three main sections: 'Hauptbeschwerden', 'Schmerzregionen', and 'Verlauf'. The 'Schmerzregionen' section contains five rows of pain assessment, each with a scale from 0 to 10 and a corresponding bar chart. The 'unter dem Kinn' row is highlighted with a red circle and the number 1, indicating it is the focus of the instruction. The patient's name and ID are displayed at the bottom of the form.

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The screenshot displays the CMDpain 4 software interface. At the top, there are menu options 'Datei' and 'Extras'. The main header features the 'CMDpain' logo and the 'dentaConcept' logo. Below the header, there are three main navigation buttons: 'Hauptbeschwerden', 'Schmerzregionen', and 'Verlauf'. The central part of the screen is a line graph titled 'Verlauf von Hauptbeschwerden und Schmerzen in den Regionen'. The y-axis is labeled 'Schmerzen' and ranges from 0 to 10. The x-axis is labeled 'Untersuchungstermine' and shows a date '16.07.2017'. A red circle with the number '1' highlights a data point on the graph. Below the graph, there is a row of checkboxes for different pain regions: 'Hauptbeschwerden', 'Schlafen/Wangen', 'Zähne', 'Kiefergelenke', 'Nacken/Rücken', and 'unter dem Kinn'. A red circle with the number '2' highlights the 'Hauptbeschwerden' checkbox. At the bottom of the interface, there is a patient information bar with the name 'Mara Musterfrau', patient number 'Pat-Nr. 123456', and the date 'Untersuchung vom 16.07.2017'. The footer of the software shows 'Patientin/Patient Mara Musterfrau Pat-Nr. 133578' and 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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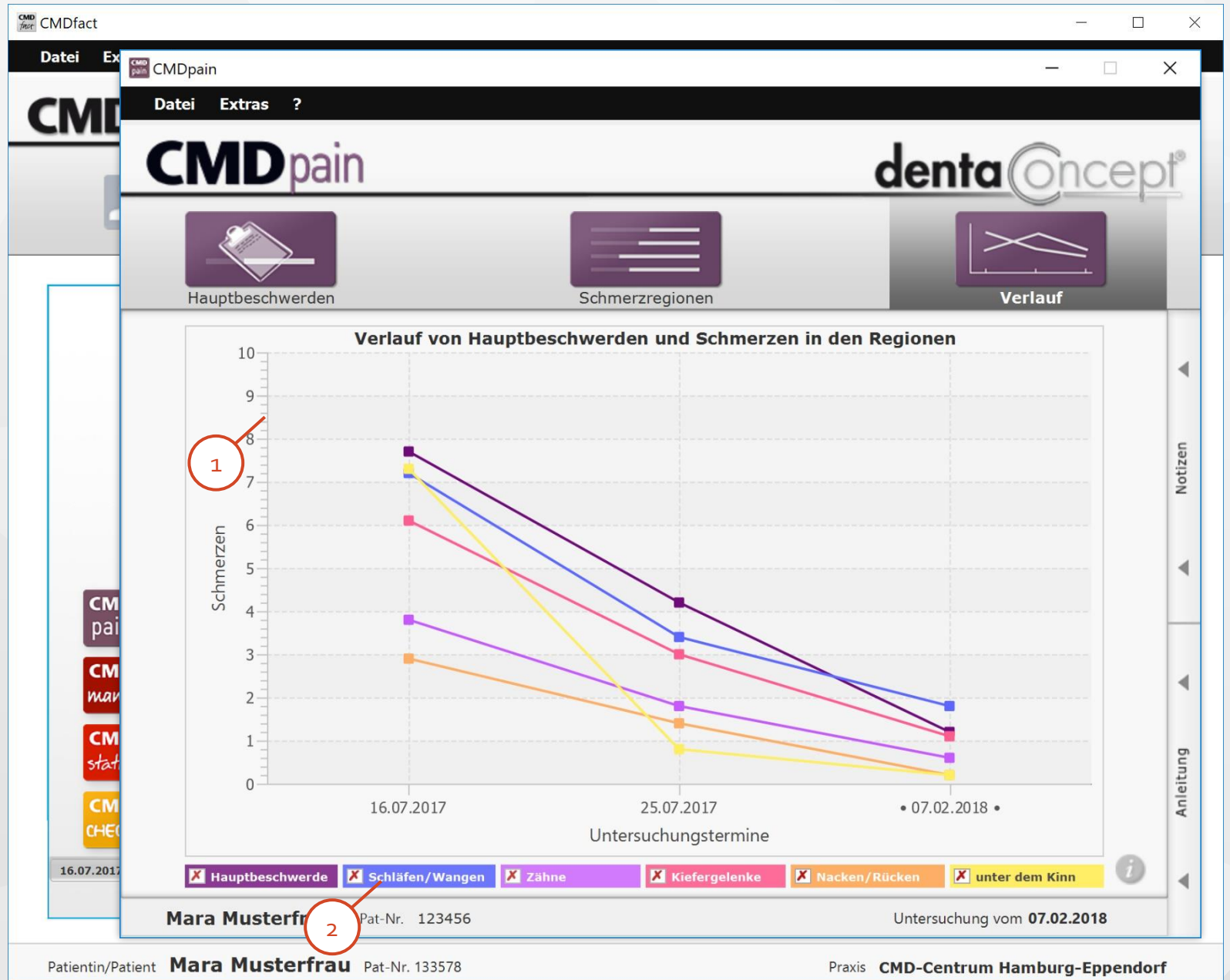
The screenshot shows the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A horizontal toolbar contains several icons: a person icon, 'CMD CHECK', 'CMD brux', 'CMD status', 'CMD manu', 'CMD pain', 'CMD tomo', 'CMD trace' (highlighted with a dashed blue box), 'CMD 3D', 'Diagnose Pilot', and 'CMD Arzt-brief'. Below the toolbar, a vertical sidebar contains icons for 'CMD trace', 'CMD status', 'CMD manu', 'CMD pain', and 'CMD CHECK'. A large blue splash screen for 'CMDtrace Version 4.10' is displayed in the center, featuring a 3D anatomical model of a human head and neck with red dots on the jaw and ear area. The splash screen also includes the text 'CMD-Centrum Hamburg-Eppendorf' and 'Priv.-Doz. Dr. M. Oliver Ahlers'. At the bottom of the interface, there is a patient information bar with the text 'Patientin/Patient Mara Musterau Pat-Nr. 123456' and 'Praxis CMD-Centrum Hamburg-Eppendorf'. A date bar at the bottom shows '16.07.2017', '25.07.2017', and '07.02'. A red circle with the number '1' is placed over the 'CMD trace' icon in the sidebar, and a red circle with the number '2' is placed over the patient name in the bottom bar. A blue arrow points from the 'CMD trace' icon in the toolbar to the 'CMD trace' icon in the sidebar. A grey arrow points from the splash screen towards the right.

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The screenshot displays the CMDtrace software interface. The main window is titled 'CMDtrace' and features a menu bar with 'Datei' and 'Extras'. Below the menu bar is a toolbar with icons for 'Instrument', 'Setup', 'Start', 'Stabilität', 'Bewegung', 'Koordination', and 'Auswertung'. The interface is divided into several sections:

- Instrument:** A section for selecting the recording system. It includes a red circle '1' pointing to the 'Registriersystem' section, which contains radio buttons for 'Zebris JMAlyser / +', 'Zebris JMAlyser fo', 'KaVO ARCUSigma', 'SAM Axioquick Recorder', 'Gamma Cadiax compact 2', 'Gamma Cadiax 4', and 'FreeCorder BlueFox'. A text box below indicates '24 Zeichen werden ausgedruckt'.
- Indikation:** A section for selecting the indication. It includes a red circle '2' pointing to the 'Prüfung der...' section, which contains checkboxes for 'Bewegungskapazität', 'Bewegungskoordination', 'Okklusale Zentrierung', 'Stabilität der Kondylenposition', and 'Artikulatorprogrammierung'. A text box below indicates '40 Zeichen werden ausgedruckt'.

The interface also includes a sidebar on the left with buttons for 'CMD trace', 'CMD status', 'CMD manu', 'CMD pain', and 'CMD CHECK'. The patient information at the bottom reads: Patientin/Patient Mara Musterfrau Pat-Nr. 123456 16.07.2017 Praxis CMD-Centrum Hamburg-Eppendorf.

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The screenshot displays the CMDtrace software interface. The main window is titled 'CMDtrace' and features a menu bar with 'Datei' and 'Extras'. The interface includes a sidebar with buttons for 'CMD trace', 'CMD status', 'CMD manu', 'CMD pain', and 'CMD CHECK'. The central area shows the 'Start' configuration window, which is divided into two main sections: 'Startposition' and 'Startkalibrierung'. In the 'Startposition' section, the 'Aktuelle Kieferposition' radio button is selected, and the 'Positionsbezeichnung' field contains 'HO (habit. Okklusion)'. In the 'Startkalibrierung' section, the 'Abweichung vom Nullpunkt gelassen' radio button is selected. Below these sections are buttons for 'rechts' and 'links', and 'Bewegungsbeginn' options for 'rechts' and 'links'. The 'rechts' 'Bewegungsbeginn' section has 'stabil' selected, while the 'links' section has 'instabil' selected. The patient information at the bottom reads: 'Patientin/Patient Mara Musterfrau Pat-Nr. 123456 16.7.2017 Praxis CMD-Centrum Hamburg-Eppendorf'.

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The screenshot displays the 'CMDtrace' software interface. The main window is titled 'Kondyläre Stabilität' and is divided into two columns: 'rechts' (right) and 'links' (left). Each column contains three rows of diagrams representing different stages of condylar movement: 'Startpunktverschiebung' (Start point displacement), 'Haltepunkt in Inkursion' (Stop point in excursion), and 'Endpunktverschiebung' (End point displacement). The diagrams show the condyle's position relative to the glenoid fossa, with labels for 'cranial', 'posterior', 'anterior', and 'caudal'. Red 'X' marks indicate deviations or stop points. The interface also includes a sidebar with 'CMD trace', 'CMD status', 'CMD manu', 'CMD pain', and 'CMD CHECK' buttons, and a patient information bar at the bottom for 'Mara Musterfrau'.

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The screenshot displays the CMDtrace software interface for movement analysis. The main window is titled 'Bewegung' and is split into 'rechts' (right) and 'links' (left) columns. Under the 'Bahnlänge' (Track length) section, the 'normal' radio button is selected for both sides. Under the 'Bahnform' (Track shape) section, the 'anterior konkav (Kurve)' and 'irregulär' (irregular) checkboxes are selected for both sides. The interface includes a sidebar with buttons for 'CMD trace', 'CMD status', 'CMD manu', 'CMD pain', and 'CMD CHECK'. The bottom status bar shows 'Demo Modus', patient information 'Mara Musterfrau', and the practice name 'CMD-Centrum Hamburg-Eppendorf'. Two red circles with numbers 1 and 2 are overlaid on the 'Bahnlänge' and 'Bahnform' sections respectively, corresponding to the numbered steps in the text.

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The screenshot displays the CMDtrace software interface. A dialog box titled "Verlauf, Koordination" is open, showing a side-by-side comparison of "rechts" (right) and "links" (left) data. The "Geschwindigkeitsverlauf" (Speed course) section has "eingipflig (normal)" checked for the right side and "mehrgipflig" checked for the left side. The "Kondylenkoordination" (Condylar coordination) section has "deutlicher Nachlauf" (clear lag) selected for both sides. A red circle with the number "1" highlights the "eingipflig (normal)" checkbox, and another red circle with the number "2" highlights the "deutlicher Nachlauf" radio button. The background shows the main software interface with various tool icons and patient information for Mara Musterfrau.

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3

The screenshot displays the CMDtrace software interface. The main window is titled 'CMDtrace' and features a menu bar with 'Datei' and 'Extras'. Below the menu bar is a toolbar with icons for 'Instrument', 'Setup', 'Start', 'Stabilität', 'Bewegung', 'Koordination', and 'Auswertung'. The 'Auswertung' icon is highlighted. A dialog box titled 'Auswertung' is open in the foreground, containing the following sections:

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At the bottom of the dialog box, there is a button labeled 'Anleitung' with a right-pointing arrow. A red circle with the number '1' points to the 'Anzeigen und übernehmen' checkbox, and another red circle with the number '2' points to the 'Anleitung' button. The background shows the main software interface with a patient information bar at the bottom: 'Patientin/Patient Mara Mustertfrau Pat-Nr. 123456 16.7.2017 Praxis CMD-Centrum Hamburg-Eppendorf'.

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Movement function of the r concept for structuring crite for standardizing computer- Expert statement for developing Diagnostic Criteria f

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Motion analysis of the mandible: guidelines for standardized analysis of computer-assisted recording of condylar movements

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- a PD Dr. med. dent. Oliver Ahlers, CMD-Centre Hamburg-Eppendorf and Department of Restorative and Preventive Dentistry, School of Dental Medicine, University Medical Centre Hamburg-Eppendorf, Germany
- b Prof. Dr. med. dent. Olaf Bernhardt, Department of Restorative Dentistry, Periodontology and Endodontology, Preventive & Pediatric Dentistry, Centre for Dental, Oral and Maxillofacial Medicine, Ernst Moritz Arndt University, Greifswald, Germany
- c Prof. Dr. med. dent. Holger A. Jakstat, Department of Prosthetic Dentistry, Dental Materials and Special Care, Center for Dental and Oral Medicine, University of Leipzig, Germany

Abstract

Mandibular movement recording has long been established as the method for the physiological design of indirect dental restorations. Condylar movement recording is the basis for individual, patient-specific programming of partially or fully adjustable articulators. The settings derived from these recordings can generally be used in both traditional mechanical and electronic virtual articulators.

- d Prof. Dr. med. dent. Bernd Kordaß, Department of CAD/CAM and CMD Treatment, Centre for Dental, Oral and Maxillofacial Medicine, Ernst Moritz Arndt University, Greifswald, Germany
- e Prof. Dr. med. dent. Jens C. Türp, Clinic for Reconstructive Dentistry and Temporomandibular Disorders, Dental School, University of Basel, Switzerland
- f Prof. Dr. med. dent. Hans-Jürgen Schindler, Department of Prosthodontics, Dental School, Ruprecht Karls University, Heidelberg, Germany
- g Prof. Dr. med. dent. Alfons Hugger, Department of Prosthodontics, Westdeutsche Kieferklinik, Heinrich Heine University, Dusseldorf, Germany

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The screenshot displays the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. Below the menu bar, the 'CMDfact' logo is on the left and the 'dentaConcept' logo is on the right. A navigation bar contains several icons: a person icon, 'CMD CHECK', 'CMD brux', 'CMD status', 'CMD manu', 'CMD pain' (circled with a red '1'), 'CMD tomo', 'CMD trace', 'CMD 3D', 'Diagnose Pilot' (highlighted with a green dashed box), and 'CMD Arzt-brief'. A green arrow points from the 'Diagnose Pilot' icon to a large green panel in the center. This panel features the text 'CMD DiagnosePilot Version 4' and '4.10' in large white font. To the left of the panel is a vertical stack of icons: 'Diagnose Pilot' (green), 'CMD pain' (purple), 'CMD manu' (red), 'CMD status' (red), and 'CMD CHECK' (yellow). A red circle with the number '2' is next to the 'Diagnose Pilot' icon in this stack. Below the panel, the date '16.07.2017' is displayed. At the bottom of the interface, the patient information 'Patientin/Patient Mara Musterfrau Pat-Nr. 133578' is on the left, and the practice information 'Praxis CMD-Centrum Hamburg-Eppendorf' is on the right.

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The screenshot displays the 'CMD DiagnosePilot' software interface. The main window title is 'CMD DiagnosePilot' and the logo 'dentaConcept' is visible in the top right. The interface features a navigation bar with five buttons: '(Initial-) Diagnosen', 'Neben- + Differential-Diagnosen', 'Diagnose-Cinema', 'Maßnahmen, Konsile' (which is highlighted), and 'Export, Drucken'. Below the navigation bar, there are two columns of checkboxes for selecting measures and investigations. The left column is titled 'Zahnärztliche Maßnahmen' and the right column is 'Konsiliarische Untersuchungen'. At the bottom of the window, the patient information is displayed: 'Patientin/Patient Mara Musterfrau Pat-Nr. 133578' and 'Praxis CMD-Centrum Hamburg-Eppendorf'. On the right side of the main content area, there are vertical labels 'Notizen' and 'Anleitung'.

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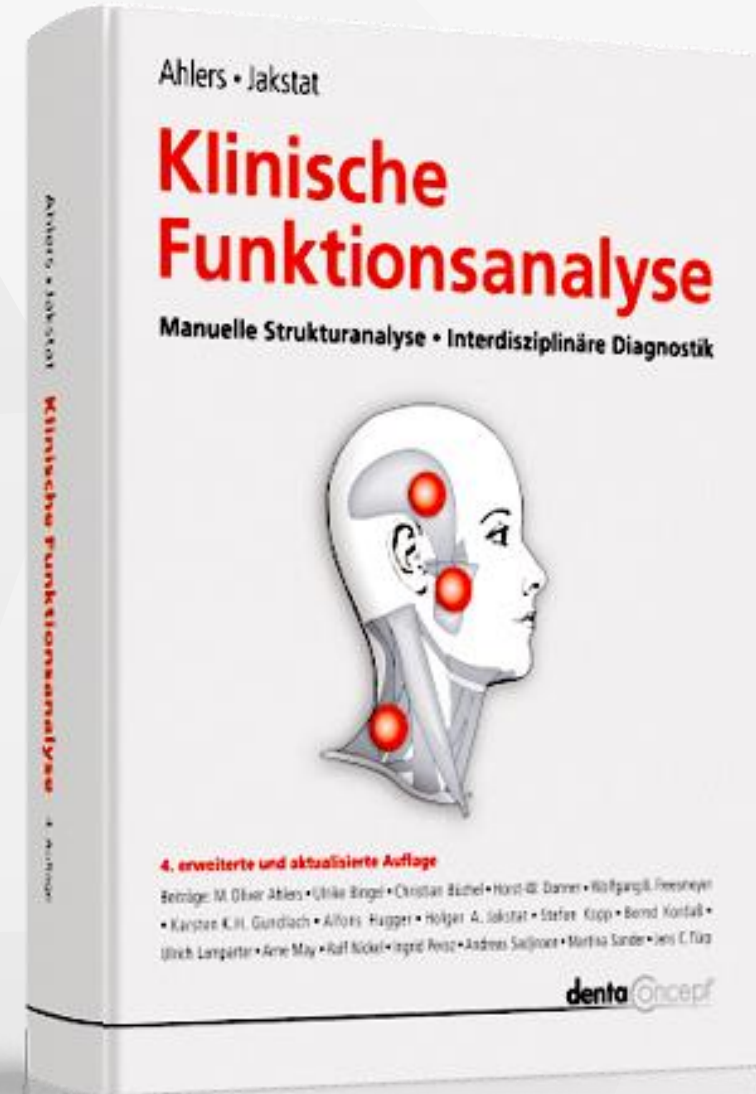
At the bottom of the software window, there is a status bar with the text 'Patientin/Patient - - - Pat-Nr. - - -' on the left and 'Praxis CMD-Centrum Hamburg-Eppendorf' on the right.

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The screenshot shows the CMDfact software interface. A dialog box titled "Praxisdaten / Aktivierungscodes" is open, displaying the following fields:

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Buttons at the bottom of the dialog are "Abbruch" and "Sichern".

The background shows the main menu with "Praxisdaten / Aktivierungscodes" selected, and the main window with "Concept" and "CMD Arztbrief" logos.

At the bottom of the window, it says "Patientin/Patient - - - Pat-Nr. - - -" and "Praxis CMD-Centrum Hamburg-Eppendorf".

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
The screenshot shows the CMDfact software interface. The window title is "CMDfact". The menu bar includes "Datei", "Extras", and "?". The "Extras" menu is open, showing options: "Praxisdaten / Aktivierungscodes", "Datenbank-Verbindung" (highlighted with a red circle and the number 1), "VDDS anmelden", "Einstellungen", "Services", and "Import von CMDfact3". The toolbar contains icons for "CMD status", "CMD manu", "CMD pain", "CMD tomo", "CMD trace", "CMD 3D", "Diagnose Pilot", and "CMD Arztbrief". The main content area features a large icon of a database cylinder and a key with a 'D' on it, with a large grey arrow pointing to the right below them. The status bar at the bottom shows "Patientin/Patient - - - Pat-Nr. - - -" on the left and "Praxis CMD-Centrum Hamburg-Eppendorf" on the right.

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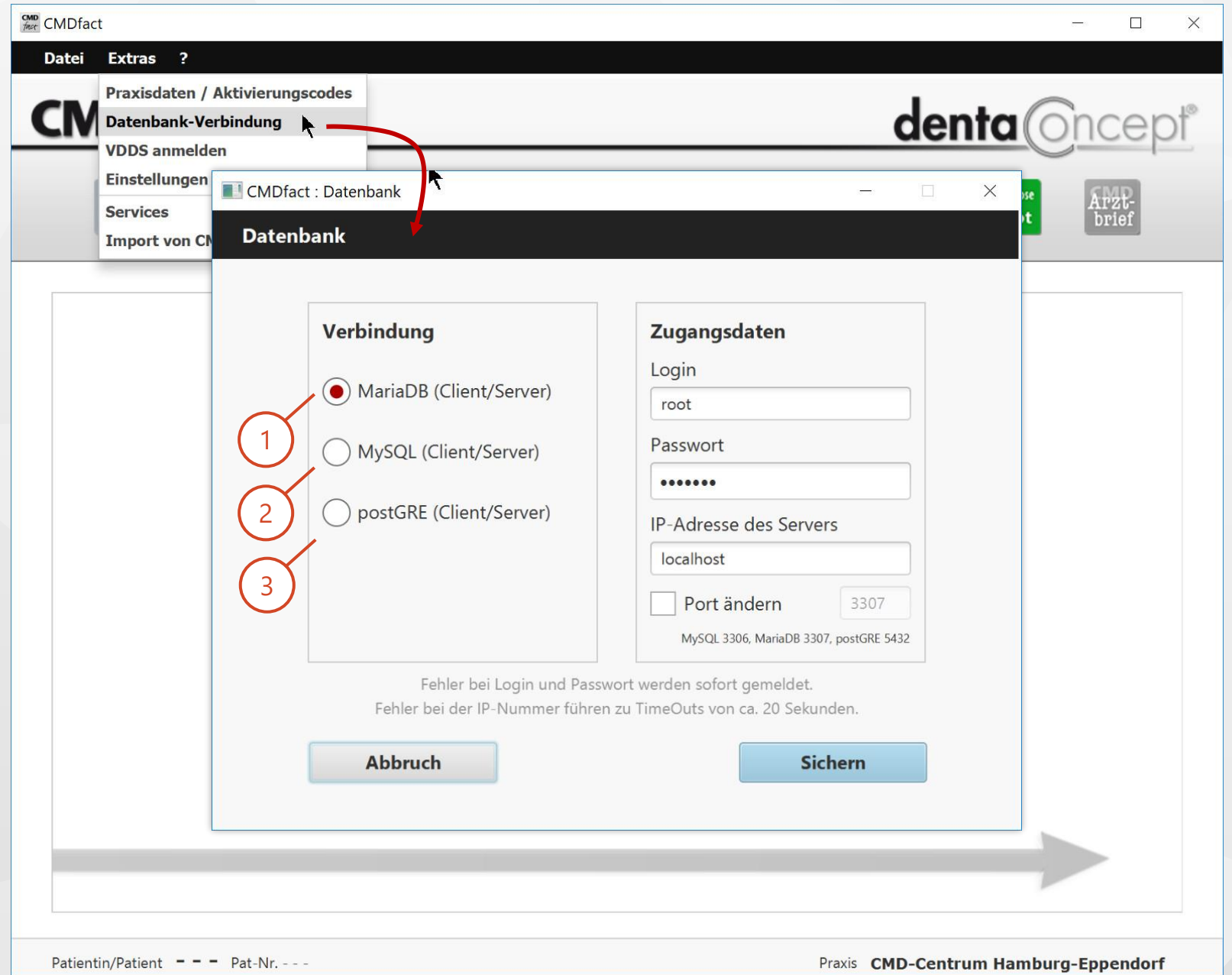Verbindung

- MariaDB (Client/Server)
- MySQL (Client/Server)
- postGRE (Client/Server)

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The screenshot displays the CMDfact software interface. At the top, there is a menu bar with 'Datei', 'Extras', and '?'. The 'Extras' menu is open, showing options like 'Praxisdaten / Aktivierungscodes', 'Datenbank-Verbindung', 'VDDS anmelden', 'Einstellungen', 'Services', and 'Import von CMDfact3'. The 'Einstellungen' option is highlighted, and a red arrow points to a dialog box titled 'CMDfact : Einstellungen'. The dialog box has a title bar 'Einstellungen' and two main sections: 'Dateiname der Ausdrucke' and 'Farbe der Messwerte'. The 'Dateiname der Ausdrucke' section includes input fields for 'Patienten-Nummer', 'Vorname des/der Patienten/in', 'Nachname des/der Patienten/in', 'Datum der Untersuchung', and 'Name der Untersuchung/des Moduls', along with a 'Muster' field containing '12345 Berger Eva CMDstatus 23-23-2003.pdf'. Below these fields are buttons for 'Neu beginnen', 'Standard', and 'zurücksetzen'. The 'Farbe der Messwerte' section contains a color selection tool with a color palette and a color code '#4d66cc'. A red arrow points to the color selection tool, and a grey arrow points from the 'Abbruch' button to the right. The bottom status bar shows 'Patientin/Patient - - - Pat-Nr. - - -' and 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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The screenshot shows the CMDfact 4 software interface. The main window has a menu bar with 'Datei', 'Extras', and '?'. The 'Extras' menu is open, showing options like 'Praxisdaten / Aktivierungscodes', 'Datenbank-Verbindung', 'VDDS anmelden', 'Einstellungen', 'Services', and 'Import von CMDfact3'. The 'Import von CMDfact3' option is highlighted with a red arrow. Below the menu bar, there are several module icons: CMD status, CMD manu, CMD pain, CMD tomo, CMD trace, CMD 3D, Diagnose Pilot, and CMD Arztbrief. The 'dentaConcept' logo is visible in the top right corner. The 'Import von CMDfact3' dialog box is open, showing a text area with instructions and a sequence of buttons: 'Patienten', 'Konflikte klären', 'KFAL-Befunde', 'CMD Kurzbefund', 'Manuelle Strukturanalyse', 'CMDtomo', and 'CMD DiagnosePilot'. A red arrow points from the 'Import von CMDfact3' menu option to the 'Import der Daten von CMDfact3' dialog box. The dialog box also has 'Import neu starten' and 'Schließen' buttons. The status bar at the bottom shows 'Patientin/Patient - - - Pat-Nr. - - -' and 'Praxis CMD-Centrum Hamburg-Eppendorf'.

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©2003 C. Runte / dentaConcept Verlag